

TechNO BOOST

Test Kit

This non-invasive saliva test allows you to instantly measure your nitric oxide (NO) levels, as well as the presence of healthy oral microflora. With this instant, personalized data you can make real-time dietary adjustments that will help to improve your cardiovascular wellness.

TEST KEY:

NITRIC OXIDE

Nitric oxide plays an essential role in your circulatory system by signaling muscle tissue to dilate and increase blood flow. Signs NO levels may be low can include poor concentration and low memory, less energy during workouts, irritability, anxiety, depression, decreased libido, poor sleep, and more.

MICROFLORA


The body uses healthy microflora (or "microbiota") to convert plant-derived inorganic NOx. Poor microflora balance symptoms include excessive gas and bloating, poor digestion, food sensitivities, abdominal pain and more.

DEFICIENT

OPTIMAL


INSTRUCTIONS:

1

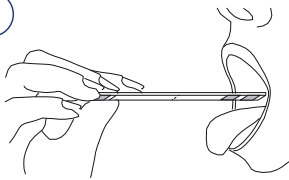

Collect saliva from tongue on collection pads for 5 seconds.

2

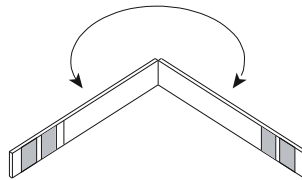


Fold strip over and gently press collection pads against test pads to transfer saliva.

3



Gently hold pads together for 10 seconds.

4

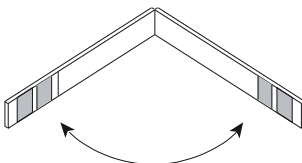

Separate pads and compare with colors on guide chart (see reverse side).

Make sure you don't eat or drink anything for an hour prior to testing. Take two capsules of TechNO Boost, wait 20-40 minutes, then test according to directions at left. Note: Age, metabolism, medication, and other factors can influence results.

This test is not intended to diagnose disease, but to help detect for the presence of the oral microflora that may convert dietary nitrates in your plant-based diet to beneficial nitric oxide.